



# Fitting QEEG Cap to the Patient

1. Begin by having the patient remove any earrings.
2. Prep the ear lobes with an alcohol swab
3. Fill the metal cups of the ear clips with Electro-Gel and attach them to the patient. Making sure that the red clip is placed on the right ear lobe and the white clip is placed on the left ear lobe.
4. Have the patient hold the front of the QEEG Cap on the forehead and stretch the cap over their head. See the pictures below for proper placement.

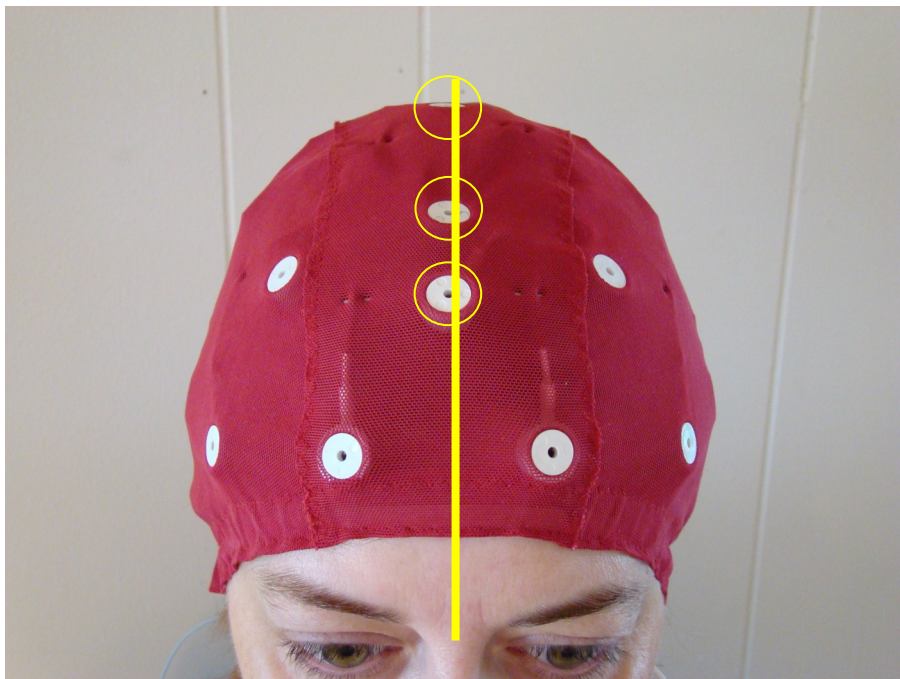


The FZ, CZ, and ground sensors should line up with the center of the patients nose.

(continued) Fitting the QEEG Cap

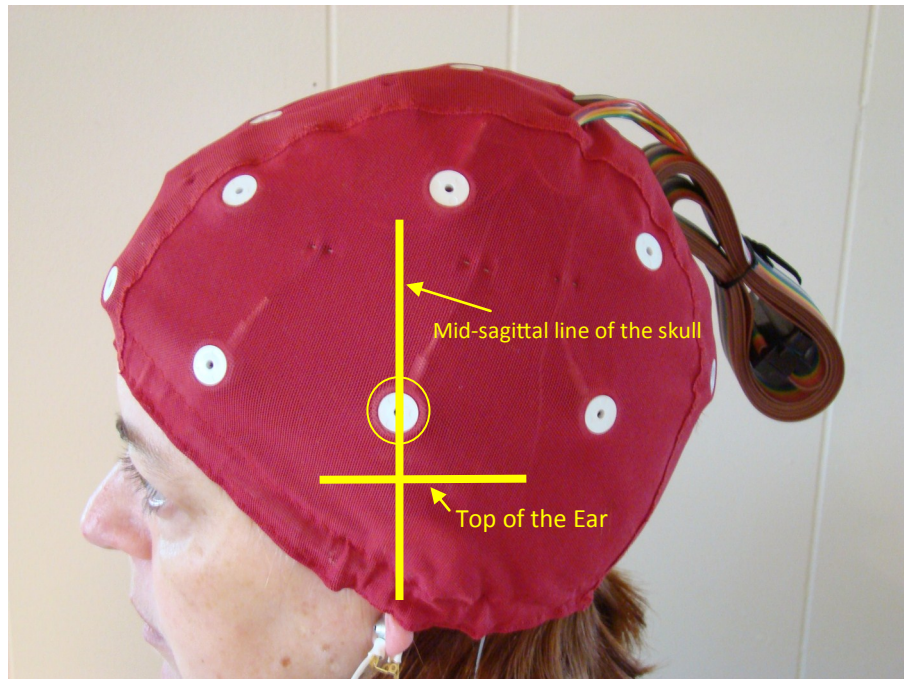

The T3 and T4 sensors should be placed on the mid-sagittal line just above the ears

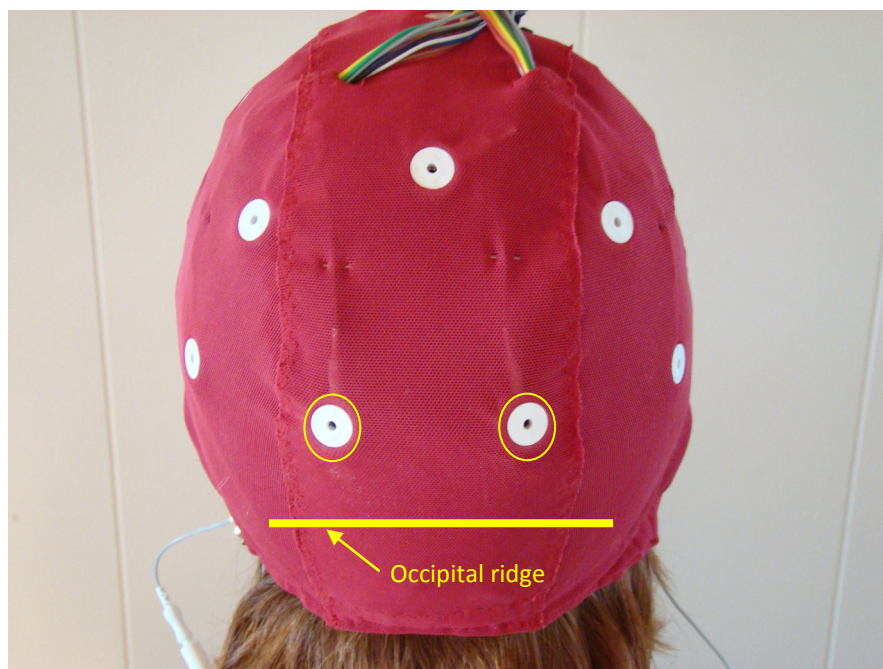

The O1 and O2 sensors should be placed 1" above the occipital ridge